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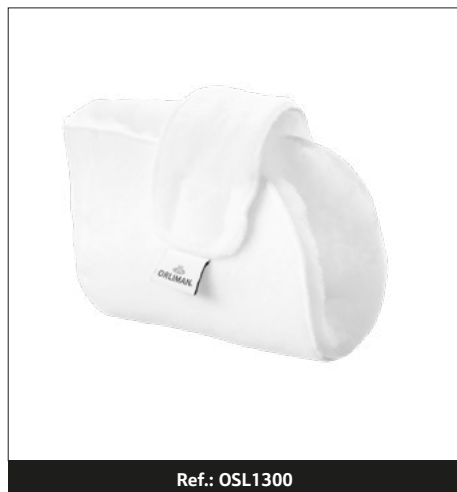
EN The manufacturing date is included in the batch number which can be found on the packaging label as , in the following way: the second and third digits represent the year of manufacture and the fourth and fifth digits represent the month.

ORLIMAN S. L. U.
C/ Ausias March, 3 - Pol. Ind. La Pobla-L'Eliana
Apdo. de correos 49 - C.P.: 46185
La Pobla de Vallbona Valencia - España (Spain)
Tel.: +34 96 272 57 04 - Fax: +34 96 275 87 00
Tel. Exportación: +34 96 274 23 33
E-mail: orto@orliman.com · Export mail: export@orliman.com
www.orliman.com

OSL1300

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Fecha de revisión/Revision date: 2022-02 | v.01

INSTRUCTIONS FOR USE, STORAGE AND WARRANTY

Dear Customer,

Thank you very much for placing your trust in an Orliman product. Please read the instructions carefully. Keep these instructions and the packaging for future reference. If you have any questions or concerns, please contact your doctor, orthopaedic specialist or our customer service department.

ORLIMAN S.L.U. guarantees all its products as long as the original configuration has not been manipulated or altered except for the intended use as described in these instructions.

If the products are used in combination with other products, replacement parts or systems, make sure they are compatible and made by Orliman®. It does not guarantee any products with altered characteristics due to improper use, defects or breakage of any kind. The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. If any serious incidents related to the product occur, notify Orliman S.L.U. and the corresponding competent authority in your country.

Orliman would like to thank you for choosing this product and hopes you a speedy recovery.

REGULATIONS

 This article is defined as a class I medical device. A Risk Analysis (UNE EN ISO 14971) has been carried out, minimising the existing risks. Tests have been in accordance with European Regulation UNE-EN ISO 22523 on Prostheses and Orthoses.

INDICATIONS

- Prevention and auxiliary treatment of pressure sores.
- People who spend long periods of time in the same position with parts of the body in contact with the surface of the bed or subjected to constant friction by bedding or rough surfaces.

Protection areas by product:

- Boot: calcaneus (heel) and malleoli.
- Elbow protector: olecranon, epicondyle and epitroclea.

FITTING INSTRUCTIONS

For best therapeutic results considering different pathologies and to extend the useful life of the product, it is essential to choose the correct size for each patient or user. Excessive compression may be intolerable; adjusting the compression to be firm yet comfortable is recommended.

If the product must be adapted, any such adjustments must be done by an orthopaedic specialist or healthcare professional legally certified to do so who must make sure the end user or person responsible for fitting the product properly understands how it works and should be used.

When fitting the product, you must adhere to the following instructions:

- 1-Undo the product's adjustable straps or fastenings and fit to the appropriate part of the body.
- 2-Fasten for a comfortable fit.

PRECAUTIONS

Before each use, check that all product components are present as per the fitting process. Periodically check the conditions of the product. If you observe any defect or anomaly, immediately report it to the issuing establishment.

This product is made of inflammable material. Do not expose the products to situations that could set them on fire. In the event of a fire, quickly get them off your body and use the proper resources to extinguish the fire.

To avoid minor discomfort caused by sweating, we recommend using some type of cotton fabric to separate the skin from contact with the product material. For discomfort such as chafing, irritation and swelling, remove the product and see a doctor or orthopaedic specialist. The product should only be used on healthy skin. It is not recommended for use over open scars with swelling, redness or hotspots.

Products marked with the  symbol contain natural rubber latex and can cause allergic reactions in people sensitive to latex.

Products marked with the  symbol contain ferromagnetic components and, therefore, extreme precaution must be taken if you undergo an MRI scan or are exposed to radiation associated with diagnostic or therapeutic procedures.

RECOMMENDATIONS-WARNINGS

The use of these products is conditioned by the indications. Although the product is not defined as a single-use device, using it on a single patient only is recommended and only for the intended purposes as described in these instructions or by a healthcare professional.

When disposing of the product and its packaging, you must strictly adhere to the legal regulations in your community.

RECOMMENDATIONS FOR STORAGE AND WASHING

Maintenance (SOFT heel protector and SOFT elbow protector): regularly brush or shake the surface of the fibre to keep it in optimum condition and maximise the product's service life.

Maintenance (TECH heel protector): wipe the outer fabric with a damp cloth. Use a dry towel to absorb any moisture.

Washing: before washing, close any Velcro fasteners or zips.

Machine washable: machine wash with delicate programme and tepid water (30°C) with neutral detergent. Spin dry gently (400 rpm). Leave to dry at room temperature. Do not use bleach. Do not hang up, iron or expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc. Do not dry clean.

When using or cleaning, do not use bleach, alcohol, ointments or liquid solvents. If the product is not well wrung out, detergent residue can remain and cause irritation to the skin and damage to the product.

Store in a dry place at room temperature.

EN COMPOSITION

- > Fabric: 90% polyester, 10% acrylic with Sanitized treatment (contains biocidal substance: zinc pyrithione).
- > Adjustable closure system: 100% polyamide.

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